

Fixodent

ULTIMATE

4 EASY STEPS

1



CLEAN & DRY

Clean and dry your denture before you apply the adhesive cream.

2



APPLY

Apply adhesive in thin continuous lines as shown or less, not too close to denture edge.

QUICK START GUIDE FOR 1ST TIME USER: Start with small amount and gradually increase amount over first week to find what works best for you.

START HERE

GOAL

Upper



Lower



Partial



READ ME

3



INSERT & HOLD

Insert denture and hold it briefly in place. If product oozes off denture in mouth, you are using too much. Keep tube, nozzle and cap dry to avoid clogging. If tip becomes clogged, use a dry pin to reopen.

4



RINSE & REMOVE

To remove your denture, rinse your mouth with warm water and rock denture gently until loose.

UK IE ADHESIVE CREAM for full and partial dentures.

IMPORTANT:

- **DO NOT** use more than shown to avoid harm such as choking. For people with reduced mental capacity, supervise use.
- **DO NOT** use product more than once a day. With proper use, this tube should last at least 4 to 5 weeks.
- **DO NOT** use excess product for poorly fitting dentures.
- **DO NOT** use more than directed. Contains zinc, consult a doctor if using other products containing zinc.

Doesn't feel right? Please consult your dentist regularly to ensure you have properly fitting dentures. Poorly fitting dentures may impair your health. Once opened, use within 6 months.

WARNING: Keep out of reach of children. Do not use in the case of hypersensitivity to ingredients. In the unlikely event of a serious suspected problem with this product consult your dentist or doctor and report this to the manufacturer and national health authority (UK: MHRA, IE: HPRA).

ULTIMATE No Flavour INGREDIENTS: Calcium/Zinc PVM/MA Copolymer (35%), Paraffinum Liquidum, Cellulose Gum (20%), Petrolatum, Silica, CI 45410, CI 15985, BHT, Tocopherol.

Made in Germany for

▲ P&G Tech. Centres Ltd, KT13 0XP, UK

UK Procter & Gamble UK, Weybridge, Surrey
KT13 0XP. Tel: 0800 731 3086.

IE 1800 409 236

Issue: 2017-11 v.2

CE 0197



UK
CA

MD



www.pg.com
21531174